




How to Measure your small pet for a Walkin' Wheels® Mini frame wheelchair

Step 1: Width	Step 2: Fold of Flank	Step 3: Length
		
<p>Simply measure across the front of your dog's chest.</p>	<p>The Fold of the Flank (FOF) is a thin piece of skin that connects the dog's upper rear leg to his body.</p>	<p>Start from the back of the front leg and measure across to the back of the rear leg.</p>

Note: An adjustment to the dog wheelchair, requiring a Philips screwdriver, is required for dogs over 6 inches high.

Because of its unique design and compact size, the [Mini Walkin Wheels](#) is easy to transport. It does not fold or come with a Tote Bag (only the regular cart is designed to fold and has the tote bag to carry it around.)